Handicare Stairlifts B.V.
Newtonstraat 35
1704 SB Heerhugowaard
The Netherlands
T +31 (0)72 576 88 88
F +31 (0)72 574 34 35
stairlifts@handicare.com
www.handicare.com

Handicare Accessibility Ltd.
82 First Avenue, Pensnett Estate
Kingswinford, West Midlands
DY6 7FJ, England
T +44 (0)1384 408700
F +44 (0)1384 408719
enquiries@handicare.co.uk

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Handicare Canada
81 Romina Drive,
Concord, ON L4K 4Z9
Canada
T 905.850.9003
F 416.260.0256

AA28094_11

Handicare 4000 User Manual

Smart seat
Simplicity seat
Style seat

EN

FR

DE

IT

ES

NL

CZ

DK

SE

NO

PT















CN

TR

AR

PL

All rights reserved. No part of this publication may be reproduced and/or published by means of print, photocopy, microfilm or in any other way or form, without prior written permission from the manufacturer. This also applies for the accompanying drawings and diagrams. The manufacturer reserves the right to change parts at any time, without prior or direct notification to the customer. The contents of this manual can also be changed without prior warning. This manual is intended for the standard model of the lift. The manufacturer therefore cannot be held liable for any damage resulting from specifications which differ from those of the standard model of the lift supplied to you. For information concerning adjustment, maintenance or repairs not provided in this manual, you are asked to contact the technical department of your supplier. This manual has been compiled with every possible care but the manufacturer cannot accept responsibility for any errors in this manual or for the consequences thereof.

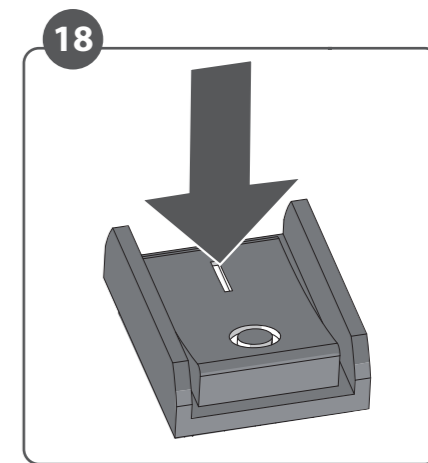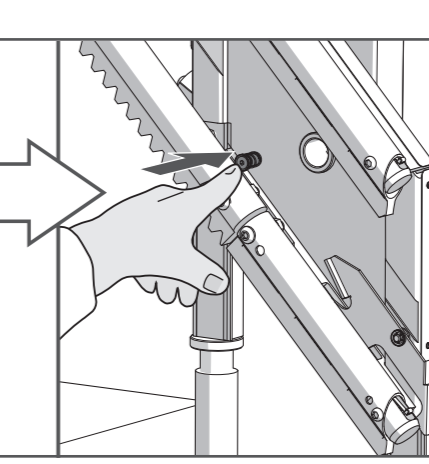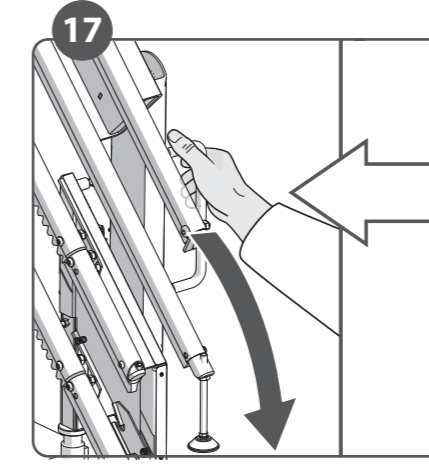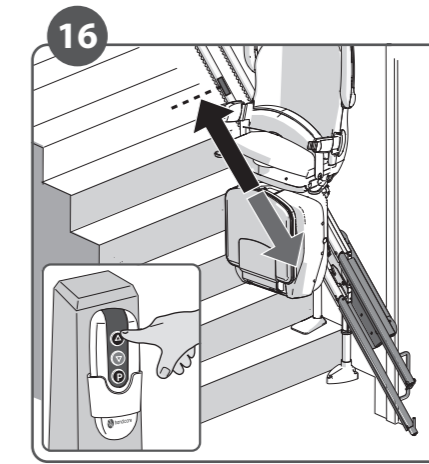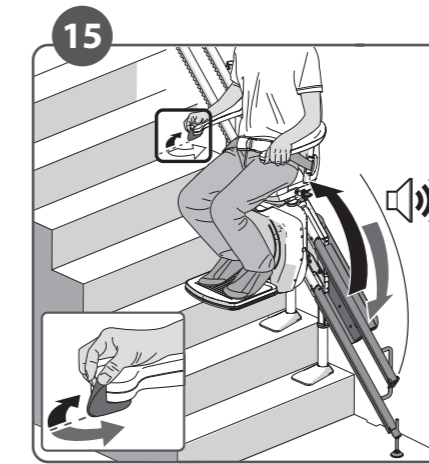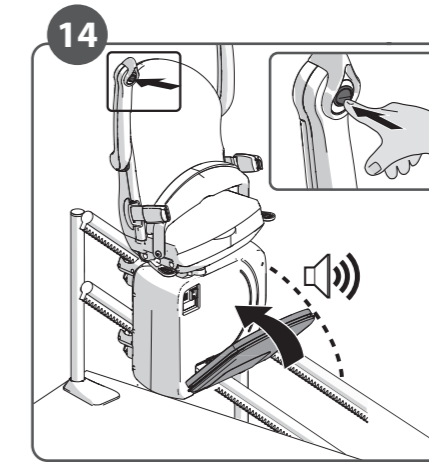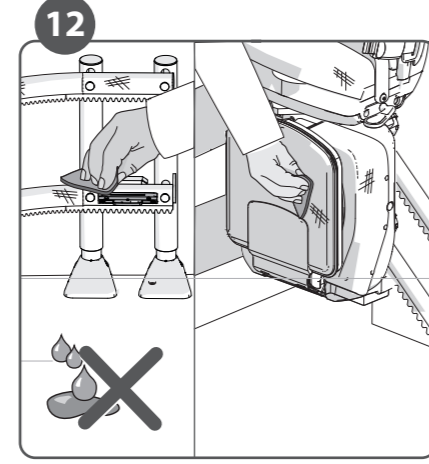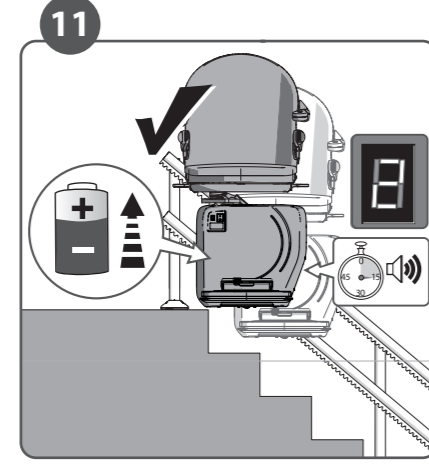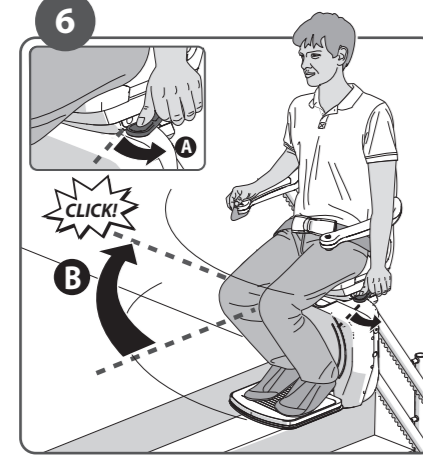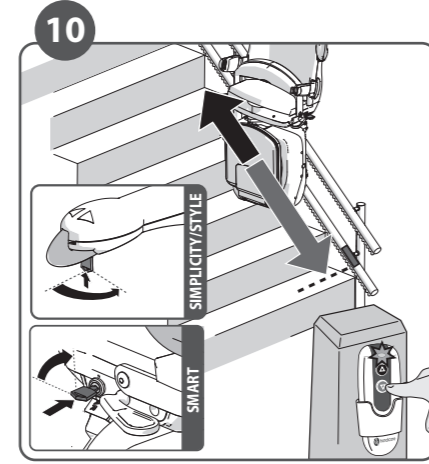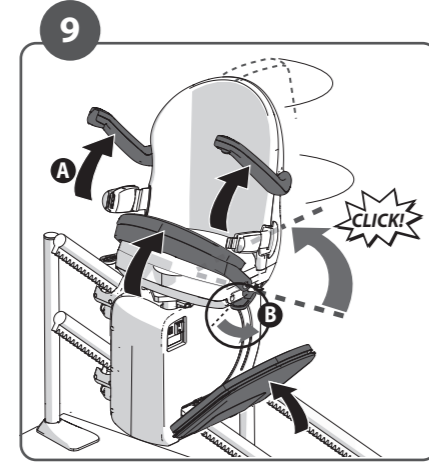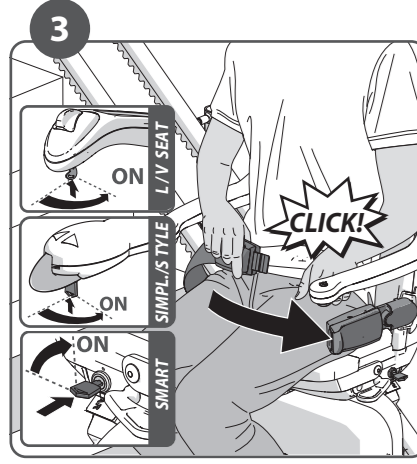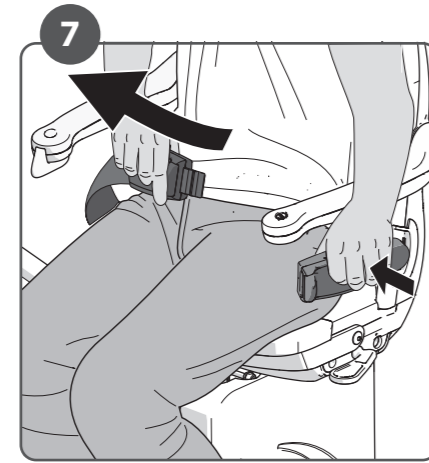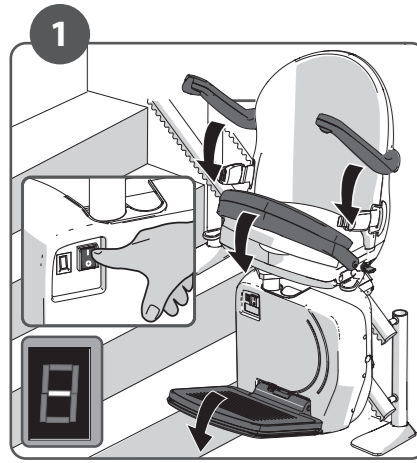ENGLISH	2	EN
FRANÇAIS	6	FR
DEUTSCH	10	DE
ITALIANO	14	IT
ESPAÑOL	18	ES
NEDERLANDS	22	NL
ČEŠTINA	26	CZ
DANSK	30	DK
SVENSKA	34	SE
NORSK	38	NO
PORTUGUÊS	42	PT
中國人	46	CN
TÜRK	50	TR
أرابيش	54	AR
POLSKIE	58	PL

1 Fold down

Ensure that the on/off switch is set in the 'ON' position. The diagnostic display will be illuminated when switched on. Fold down the footplate, the armrests and seat. *(Optional powered footrest > 14)*

2 Place feet

Place your feet on the footplate. Ensure that both feet are completely on the plate.

3 Safety first

Fasten the seat belt securely, ensuring that it clicks into the holder. Put the key in the switch and turn it ninety degrees clockwise. Check that the chair is in the locked position.

4 Ride

Move and hold the toggle in the desired direction of travel. The display shows a "8/9", the lift will start slowly after 1 second. When changing the direction of travel the delay will be 3 seconds.

5 Stop button

In an emergency, press the stop button once. The lift stops straight away. The stop button can be released by pushing the stop button.

6 Swivel

The lift stops automatically at the end of the rail. You should now swivel the seat manually, making it easier to get on and off. To do this, push down the lever{A}. Swivel the seat until you hear a 'click'. The seat is now in the correct position{B}. *(Optional autoswivel > 13)*

7 Loosen belt

Unfasten the seat belt.

8 Stand up

Stand up from the seat.

9 Fold up

Fold up the armrests, footplate and seat {A}. Push the handle, located to the left or right under the seat, downwards {B}. Swivel the seat until you hear a 'click'. The seat is now in the locked position.

10 Park

Press and hold the button on the remote control until the lift reaches a charge point. Upon arrival, the lift stops automatically.

11 Charge

There are charge points for recharging the lift's batteries at the top and bottom of the rail (there may be additional charge points). If the lift is not in the correct position at the charge point, the lift beeps for 15 seconds and the display shows a "2". The lift does not work if the batteries are flat.

12 Cleaning

Clean the stairlift regularly with a damp cloth; avoid excessive use of water. Never use an acidic cleaner. Ensure the track is kept free of dirt and dust. Over time the track wheels may leave a mark on the rail.

OPTIONS

13 Autoswivel

The AUTOMATIC SWIVEL SEAT option let you swivel the seat automatically to face the direction you are going to walk. To do this, push and hold the joystick in the direction of travel.

14 Powered footrest

To operate a POWERED FOOTREST press the switch under the armrest. The stairlift should beep whilst folding the footrest. Always ensure the footplate is in the down position during travel.

15 Hinged rail

With the FOLDING HINGE option, the seat stops just above the hinged rail. Move the joystick up or down to fold or unfold the rail.

16 Hinged rail (charging)

Do not leave the lift on the hinged rail after use. Use the remote control to move the lift to a charge point and recharge the batteries. The first stopping point is a charge point.

17 Hinge emergency operation

If you are using the stairlift and the hinge system fails, please return to upper landing and call for assistance. Do not attempt to disembark mid-travel, only disembark at the upper or lower floors. To fold the hinge manually In case of emergency: push the release button and move the main arm at the same time.

Tip: Loading of the main arm during normal operation can cause the pinion button to appear to be resistant to movement. Moving

the main arm at the same time as pushing the release button will reduce the pressure on the pinion and make movement easier.

18 On/off (option)

In case of power-out in the building, switch the power off from the lift to avoid battery drainage.

Check these points in the event of a malfunction:

1. Is the seat in the correct position before departure?
2. Are seat and armrests down?
3. Is the elevator on?
(key switch)
4. Has the emergency stop been activated?
5. Is the seatbelt closed?

In all other cases contact your supplier.

FOREWORD

Your Handicare 4000 stairlift is a sophisticated product that has been specifically developed to carry one person (max 136 kg/21.4 stone/300 lbs) sitting on the seat (heavy duty type: max 160 kg/25 stone/352 lbs). Before using your Handicare 4000 stairlift please read this manual thoroughly to familiarise yourself with how to operate your stairlift.

Your stairlift has been constructed with state of the art technology and in accordance with recognised safety regulations so you can be sure that you have a safe and reliable solution.

Nevertheless, incorrect use of the stairlift may cause risk to the user or third parties, damage to your stairlift or other goods. Use your stairlift only when it is in good working order, in accordance with the intended use described above and with due attention to this manual. Always keep this manual near the stairlift.

SAFETY INSTRUCTIONS

When using the stairlift always observe appropriate safety regulations regarding risk of electric shock and physical injury.

1. Provide an unobstructed environment by ensuring that the entire track is free of obstacles.
2. Take environmental influences into account. Do not expose the lift to damp or extreme sunlight. Make sure that the environment is well lit. In case of outdoor lift: always put the cover over the lift when the lift is not in use.
3. The stairlift should only be used by individuals who have received the appropriate guidance. On installation the supplier is required to demonstrate how to use this stairlift and the user is required to sign the declaration confirming that this instruction has been received.
4. Stairlifts should only be used for carrying one person: do not use your stairlift to carry pets, goods or more than one person at a time, even if the load is below the maximum user weight. Keep children and pets away from the stairlift.
5. Remove the key after use to prevent unauthorised use.
6. Keep limbs and clothing away from moving parts. Make sure that wide or loose clothing cannot become trapped between the chair and the stairs or rail.
7. Ensure a safe position during use. Always ensure a correct, stable position. Please sit as far back in the seat as you can and place your feet as close to the centre of the footplate as you are able to. Always use the safety belt and never stand on the stairlift.
8. Take care at all times. Concentrate on the operation of the stairlift as releasing the operating toggle will cause the lift to stop. To restart the lift wait 3 seconds and push and hold the toggle in the direction of travel desired until you reach the landing or hallway.
9. Check for damage. Before use check the lift for external damage. Report unusual noises, such as squeaking and creaking, to your supplier. To avoid accidents repairs and preventive maintenance should only be carried out by engineers qualified to do so.
10. The stairlift must not be used in the event of a fire or as a means of escape during a fire.
11. If the stairlift is not fitted with a communication device, we recommend that you always have one to hand.
12. Only leave the stairlift at the designated points at the top and bottom of the rail.

Structure

The strength and the suitability in general of wall, staircases, terraces, upper floors, banisters, plinths etc. which are to support or surround the installation have been verified and ensured by the customer, who therefore accepts all relative responsibilities. Neither Handicare nor its authorised dealers may be held responsible for the suitability of the structures which surround and support the installation.

MAINTENANCE

Cleaning

Clean the stairlift regularly with a damp cloth; avoid excessive use of water. Ensure the track is kept free of dirt and dust. Over time the track wheels may leave a mark on the rail.

Preventive maintenance

In addition to cleaning your stairlift, periodic preventive maintenance is necessary for continued long-term use.

Your stairlift must be serviced at least once a year by a qualified person. If your stairlift has more than two users we recommend a minimum of two services per year.

Please make an electrical socket available to provide local lighting if required during inspection and servicing. Lighting to a minimum value of 50 lux shall be provided at the boarding points whilst the stairlift is in use.

Most light bulb packaging will detail the lux value of the light bulb.

GUARANTEE AND LIABILITY

Term of warranty – this warranty is valid for a period of 24 months from the date that the product ships from our factory. The purchase price for the product must be paid in full in order for the manufacturer to release parts under this warranty.

Coverage – this warranty applies to the repair or replacement, at the manufacturer's discretion, of parts that fail due to defective material or workmanship. The manufacturer may, at its discretion, provide factory reconditioned parts. This warranty is provided to the Authorised Handicare Dealer on behalf of the final purchaser of the product and is not transferable. The manufacturer's warranty does not cover labour charges for the removal, repair or replacement of warranty parts.

Damage caused by normal wear and tear, overloading or misuse and the replacement of batteries in the remote control is not covered by this guarantee.

We recommend that your stairlift is serviced annually by an appropriately qualified stairlift engineer.

Under no circumstances can the manufacturer accept responsibility for death, personal injury, damage to property, or incidental, unforeseen consequences and/or costs or damage resulting from the use of your stairlift.

The carrying out of alterations to the product is not permitted and doing so will invalidate the warranty. The manufacturer is not liable for damage resulting from such alterations.

The manufacturer accepts no liability for damage or injury caused by the non or inadequate observance of the safety regulations and instructions in this manual, or by negligence during the installation, use, maintenance and repair of this stairlift.

Installation and servicing should only be carried out by dealers or installers/service engineers authorised by the manufacturer in accordance with local regulations.

The wall mounted remote controls operate on a radio frequency; Handicare is unable to accept responsibility for system inoperability in the unlikely event of interference from signals external to this stairlift.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy, and if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment OFF and ON, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

TECHNICAL SPECIFICATIONS

Supply voltage	230 VAC	Speed	0,12 m/s (max.)
Frequency	50/60 Hz	Ambient temperature	0 - 40°C
Current	1 A	Relative humidity	20 - 80%
Power 136 kg	360 watt	Usage	16 rides per 24 hours
Power 167 kg	400 watt		10 rides continuous
Fuses	30 A	Noise	56 dB
Load	(std.) 136 kg (300 lb) (h.d.) 160 kg (352 lb)		

The information provided in this manual is based on general specifications regarding the construction, material properties and working methods at the time of publication.

The configuration of the Handicare stairlift, including in particular (but not limited to) its parts such as the rail system, the construction of the stairlift, the fixation parts, all swivelling and movable parts, and the motor, rely on Handicare's know-how. Work on this configuration or on one or more of its parts may only be performed by dealers (and their employees) who have been certified by Handicare to perform such work in accordance with the instructions given by Handicare. Only these employees have received the required training; only these qualified employees can provide the level of safety that Handicare stairlifts have when delivered ex works. Suppliers of used Handicare stairlifts other than dealers who have been certified by Handicare are not qualified to perform such work, e.g. installing a rail for a user other than the original Handicare rail.

This is important, because when work is performed on a Handicare stairlift or a used Handicare stairlift is re-installed with other parts, such as a different rail, by anyone other than the aforementioned qualified employees, Handicare's warranty and the CE mark for the stairlift in question will become invalid with immediate effect.

Once the CE mark has become invalid, this means that the stairlift in question no longer complies with European regulations. As a result, Handicare will no longer be responsible or liable for any consequences arising from the failure of the stairlift in question. Furthermore, maintenance on the Handicare stairlift may only be performed using original spare parts that have been approved by Handicare.

This is important, because if parts other than original spare parts approved by Handicare, such as a different rail, are used, the Handicare warranty and the CE mark for the stairlift in question will become invalid with immediate effect.

The Dutch manual is the original language.

EN

DECLARATION ENGINEER	
I/We certify that onI/we thoroughly examined this lift and that the foregoing is a correct report of the result. I/We declare that the erected stairlift is compound with approved components and installed according to the instructions in the installation manual. I/We also declare that I/we have instructed the user by telling him/her all users practice. The user is independently able to use the lift safely without further instruction. The user manual has been handed over to the user.	
Name Engineer: Qualifications: Signing date: Signature:	Name Engineer: Qualifications: Signing date: Signature:
Address:	If employed by a company/association>name/address:

DECLARATION INSTRUCTED PERSON	
Herewith confirms the user of the stairlift that the above mentioned engineer(s) has declared all users practice and that he/she is known with the complete operation as instructed person.	
Name :	Address:
City:	Phone number:
Signing date:	Signature:

EC DECLARATION OF CONFORMITY		
We, Handicare Newtonstraat 35, 1704 SB Heerhugowaard declare that the Handicare 4000 is in conformity with the provisions of the Machinery Directive 2006/42/EC, Low Voltage Directive 2014/35/EU and EMC Directive 2014/30/EU.		
The stairlift has been designed and manufactured in accordance with the following standards: EN 81-40, ISO 9386-2, EN ISO 12100-1, EN ISO 12100-2, EN ISO 14121, EN60204-1 (including the EMC standards).		
Heerhugowaard, The Netherlands, 15 april 2023.		
P. Slack, Vice President, Operations	Year of build: 2023	Date put into service:

The declaration of conformity is legal after the installation checklist and the declaration instructed person have been signed off by the engineer and the user. The installation checklist and the declaration instructed person shall be saved by the dealer and made available to Handicare on request.

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EC DECLARATION OF CONFORMITY

We, Handicare Newtonstraat 35, 1704 SB Heerhugowaard declare that the Handicare 4000 is in conformity with the provisions of the Machinery Directive 2006/42/EC, Low Voltage Directive 2014/35/EU and EMC Directive 2014/30/EU.

The stairlift has been designed and manufactured in accordance with the following standards:
EN 81-40, ISO 9386-2, EN ISO 12100-1, EN ISO 12100-2, EN ISO 14121, EN60204-1 (including the EMC standards).

Heerhugowaard, The Netherlands, 15 april 2023.

P. Slack, Vice President, Operations, Handicare Year of build: 2023 Date put into service:

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